

LimbCoat®


A waterproof cover to protect bandages, casts and dressings from getting wet when bathing, showering or swimming.

Measurements


There are two measurements that are required to select the most suitable LimbCoat for the arm. Both measurements to be taken in centimeters.

The Arm

- 1. Circumference:** At the point of the measurement of **3cm to 5cm** above the bandage, cast or dressing, measure the circumference of the arm at this point.

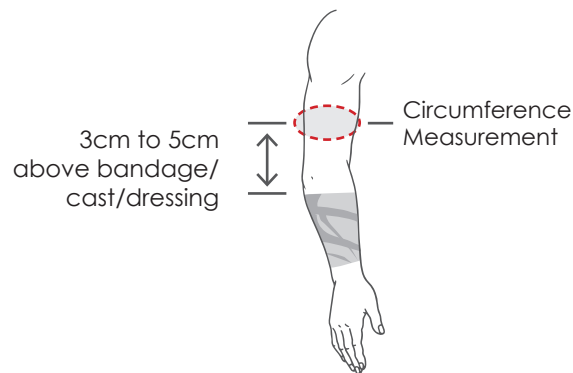

- 2. Length:** Measure from the point of where the circumference measurement was taken to the end of ones fingertip.

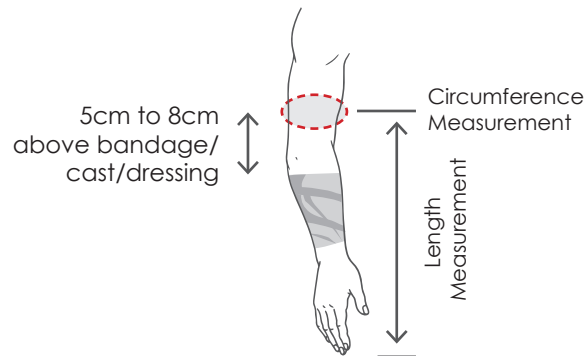

Useful Information to Assist in Measuring up Ones Arm

The circumference measurement must be located on the skin of muscle or fleshy skin area, so as to ensure a watertight seal.

A watertight seal depends on the point of the arm where the circumference measurement has been taken.

A circumference measurement located on a boney area or boney prominence (avoid ankles, elbows and knees) will compromise a watertight seal.